



# Installation

## Important

Read these instructions carefully and completely before installing and using the Aivia..


**Never install the Aivia where it is exposed to direct sunlight. You may expose the defibrillator to excessive temperatures.**

## Presentation of the Aivia

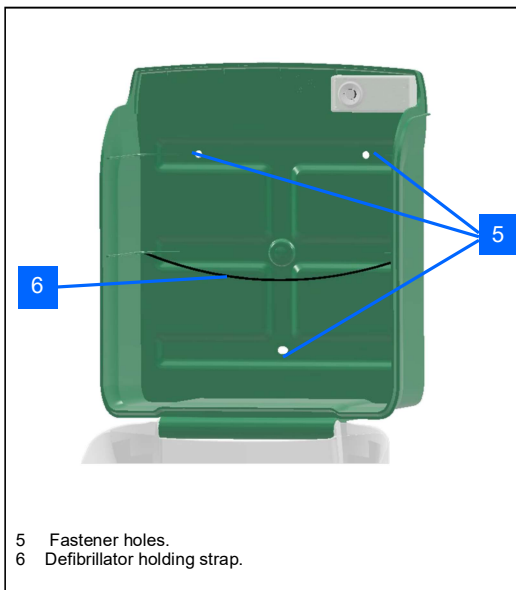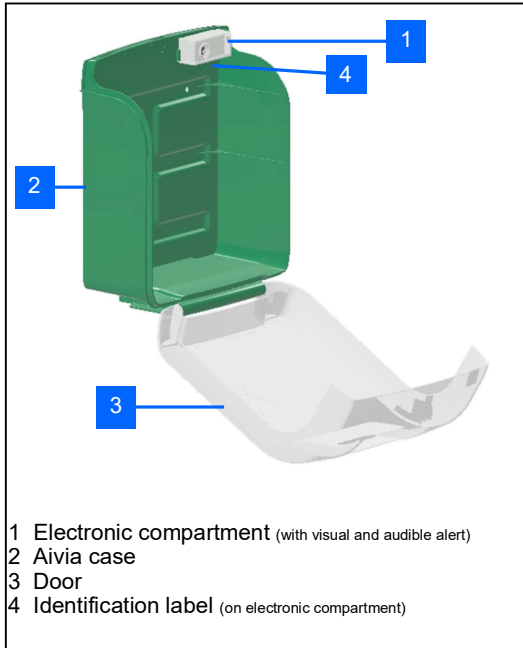

## Identification label


• The identification label is affixed to the inside of the Aivia.

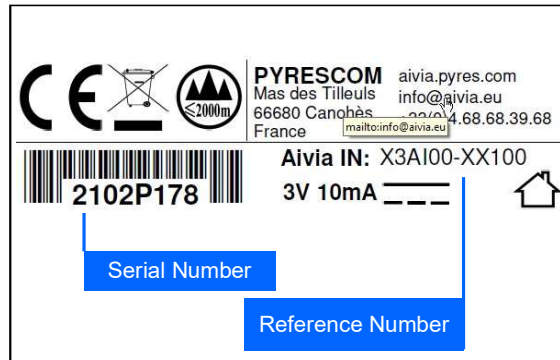

• For any correspondence with Customer Service or your service provider, please have the Aivia's reference number and serial number.

## Initial opening

• To open the Aivia IN before it's attached to the wall, hold the Aivia's case and push on both ends of the door.

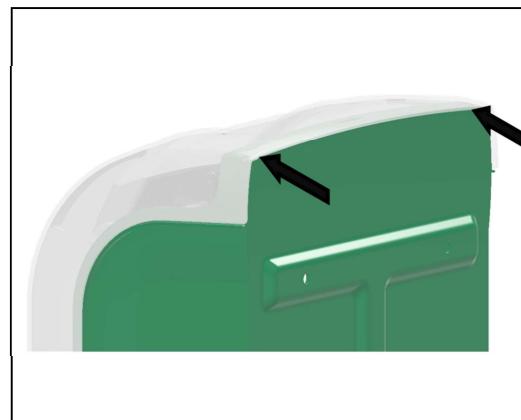


# Installation


## Installing the Aivia

**Never install the Aivia directly exposed to sunlight. The Aivia must be installed in a temperate environment in accordance with the defibrillator manufacturer's instructions.**

- Must be installed by a qualified technician or one authorised by the manufacturer of the Aivia and in adherence to the Bornavie Charter.
- If the Aivia is installed in a public area or passageway, provide a floor area (abutment) or a column to comply with the Order of 15 January 2007 "Technical requirements for free passageways" Article N°6-D.
- The fasteners must be appropriate for the type of support on which the Aivia is mounted.
- Leave free space for the Aivia as specified in the diagram below.
- The Aivia's specific dimensions are provided at the end of this document in the specifications section.

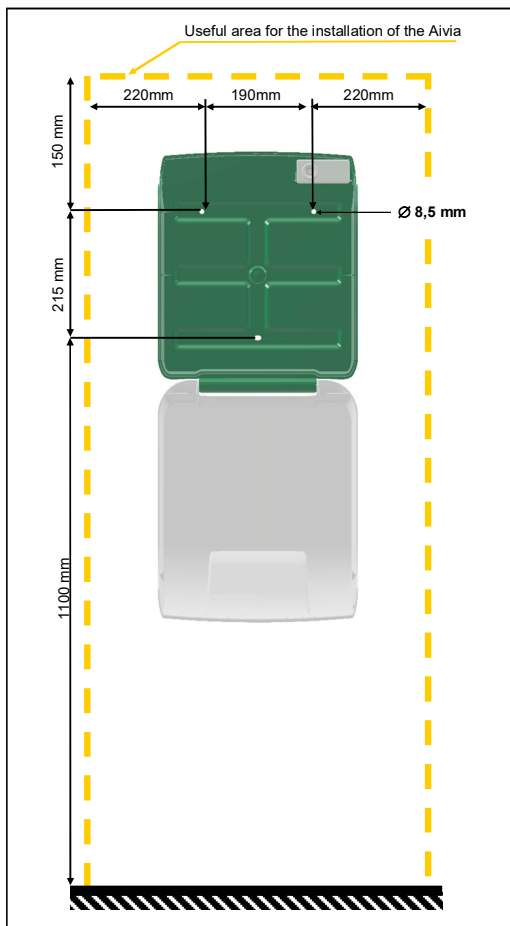


- Leave a space of 1.40 metres from the Aivia's wall bracket so that the door can be opened easily.

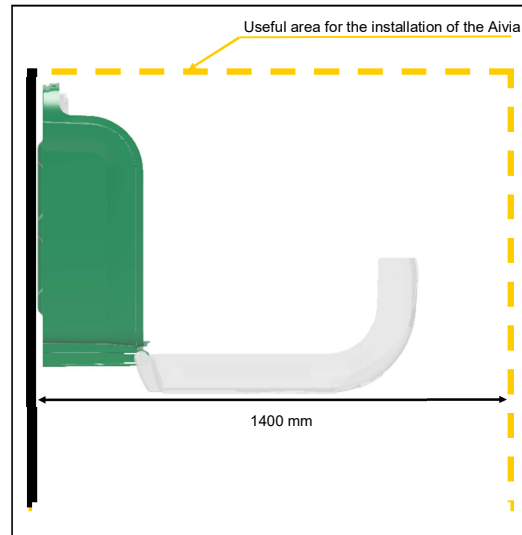

Centreline mounting distance to be observed




- The door can be unhooked to use the box as a mounting template.

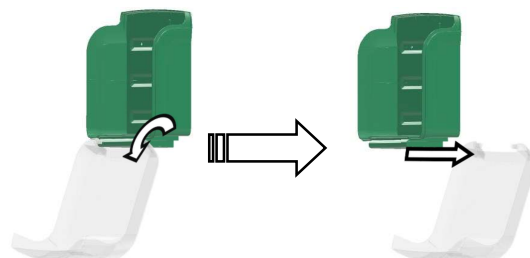

# Installation

- Use a mounting kit suitable for the type of support on which the Aivia is installed.
- The mounting must be able to withstand a minimum load of 20Kg.

• Mounting example:

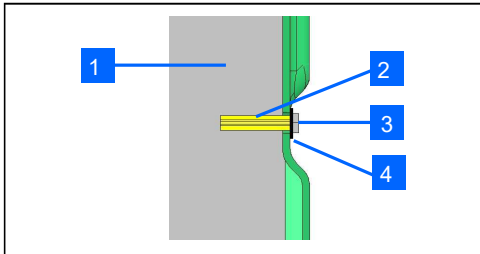

- 1 Support.
- 2 Wall plug.
- 3 Mounting screws: M6 minimum, M8 maximum.
- 4 Washer plate Ø 14mm maximum.

• The screw/washer assembly must not exceed a thickness of 8mm.

**Attach the Aivia via the three mounting holes.**

## Contents of the box



- Take out and open the box containing the additional items

## Additional items



- 1 Electronics compartment (see page 6 for use)
- 2 AA batteries (see page 6 for use)
- 3 Seal labels (see page 7 for use)
- 4 Holding strap (see page 7 for use)

## Setting up the Aivia IN

- Place the 2 AA batteries in the electronic casing, observing the correct polarity.



- Clip the electronic compartment onto the box using an upward motion.



- Check that the Aivia is switched on when the door is closed.

*Alert lights blink red twice*




*The LEDs did not flash (see page 9)*

- The Aivia is operational.
- When the door is opened, the visual and audible alarm will be activated for 30 seconds.

# Installation

## Presentation of additional items

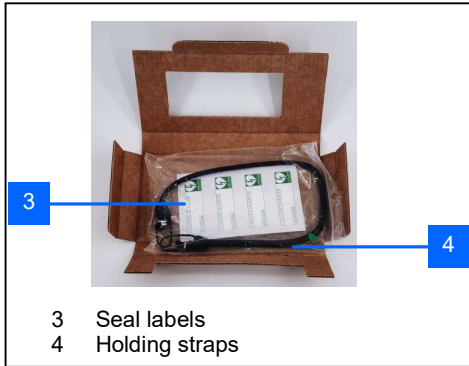


## Insertion of the defibrillator

- Hook the strap in the holes provided for this purpose.



- Pull the defibrillator holding strap and slide the defibrillator to the bottom of the Aivia.

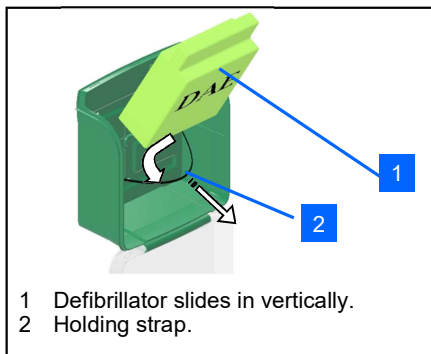

*The holding strap allows for easy removal in an emergency while holding the defibrillator in place.*

## Installation of the seals

- The Aivia comes with two sets of seals.
- The seals alert you to the opening of the door.
- Regularly check the condition of the seals.


## Installation of the seals

- Close the Aivia's door completely. Make sure to pull the foot of the door towards you before rotating the door

- The surfaces of collage must be clean, dry, fat free.



- Remove the protective film from the seal.

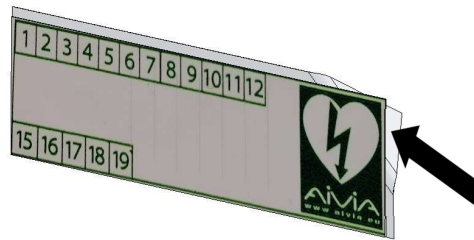

- Stick the seal across the Aivia's door and frame.




- Place one seal on each side of the Aivia.



## Pictographs

- The signage on the door of the Aivia shows you what to do in case of emergency.
- Under SOS, you must include the number to call in case of emergency if it is not listed there.

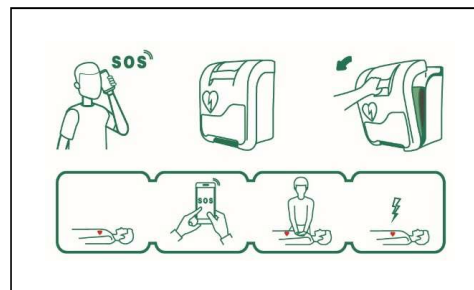

# Usage

---

## Operation of alerts


- Upon opening the door, a visual alert is given via the flashing of the red lights.
- The audible alarm is activated for 30 seconds when the door is opened.

---

## Use of the Aivia


### If you witness an incident requiring the use of a defibrillator.

- **Contact emergency services.**  
Speak in a loud and clear voice, describe the situation and specify the location of the incident.

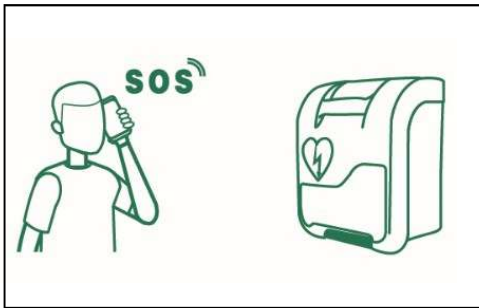

- **Pull on the Aivia's door to open it.**

A visual alert is given via the flashing of the red lights. If your Aivia is equipped with an audible alarm, it's activated for 30 seconds).

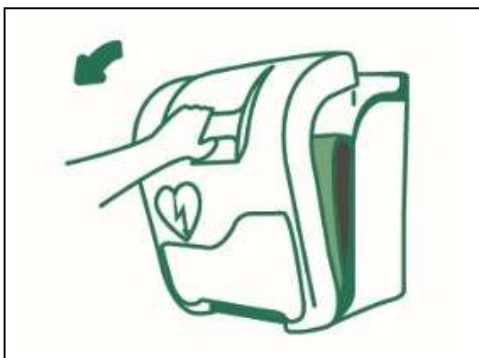

- **Grasp the defibrillator.**



- **Approach the victim.**
- **Activate the defibrillator.**
- **Follow the defibrillator's instructions.**

---

## After using the defibrillator

- Do not put the defibrillator back in the Aivia until it has been reactivated.
- Follow the instructions in the defibrillator's manual for putting it back into service.
- Your provider can help you put the defibrillator back into service. Please do not hesitate to contact him or her.
- When the defibrillator is back in service, install it in the Aivia following the instructions in the "Setting up the defibrillator" section.
- Install the seals by following the instructions in the "Installing the seals" section.

---

## Maintaining the Aivia

- Clean the surfaces of the Aivia with a soft, damp cloth. Wipe the Aivia with a clean, dry soft cloth. Do not use chemicals or cleaning products, as this could damage the Aivia.
- Do not rub the Aivia with a hard object, as this may scratch or permanently damage its surface.
- Do not use a high-pressure washer.
- Regularly check the condition of the seals.
- If the Aivia's seals have been compromised, make sure that all defibrillator components are present and in good working order. If in doubt, contact your defibrillator dealer.